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Fax: 031-376-5993

Asan EasyTest Yeast

One step detection and presumptive identification for Malassezia and candida

● APPLICATION

Asan EasyTest Yeast is the selective medium for the differential color diagnosis of Malassezia and Candida.

● PRINCIPLE OF THE TEST

The principle of this test is the color change of the medium as followed:

- M. pachydermatis- brick red with rough whitish edges
- C. albicans- green to blue green
- C. krusei- light pink with rough white border
- C. tropicalis- dark blue with purple halo around colony
- C. glabrata- dark rose
- C. lusitaniae- no change color

● TEST PROCEDURE

1. If the disease skin is contaminated, please clean the skin using non-anti-microbial shampoo and then dry the site.
2. Identify and select the pathogenic skin or horny substance. Wood lamp positive or damaged wool is also one of good specimen for this test.
3. Take out the wool from the pathogenic site using forceps. It does not need lots of wool for this test.
4. Open the cap of Asan EasyTest Yeast and then carefully inoculate the wool or horny substance onto the medium.
5. Close the cap but do not titty close (ventilation is required). And then, mark the name, number, date, and other information of the sample.
6. Culture the inoculated Asan EasyTest Yeast in the shade condition at room temperature.
7. Check and observe the color of medium at every 24 hours.

● INTERPRETATION


Candida generally change the color within 24 hours to the one described in "PRINCIPLE OF THE TEST". Malassezia normally change the color at 48~72 hours.

● PACKAGE

- 10 Tests/kit or others

● STORAGE & EXPIRATION

1. Asan Easy Test Yeast should be stored at between 2~8℃ (35.6~46.4°F)
2. Expiration date of this kit is 12 months after its manufacture date.

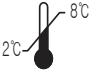● CLAIMS

This product is manufactured under our strict quality control. However, If some claims occurs, please contact us (Tel +82-32-376-5990)

REF Code No.: 23111, 23115
Document Code: AYEAO0-D0K00

ISO13485:2003

Manufactured & Sold by

ASAN PHARMACEUTICAL CO., LTD

Head Office: 163, Yeongcheon-ro, Dongtan-myeon, Hwaseong-si, Gyeonggi-do 445-813, Korea
Factory: 122-26, Giupdanji-ro, Gongdo-eup, Anseong-si, Gyeonggi-do, 456-821, Korea

Tel: +82-31-376-5990~2

Fax: +82-31-376-5993

<http://www.asanpharm.com>